



R-shield

Stylish scarf



LIFE'S WORTH IT

Instructions for use:

1. Pull the scarf carefully over your head
2. Adjust the scarf so that the nanofiber filter is in front of your mouth
3. Press the adjustable nose clip over the bridge of your nose in order to prevent air leakage around your nose
4. Use the elastic band with a plastic cord to hold the scarf firmly around the head
5. Make sure all the inhaled air goes through the nanofiber membrane
6. After use, loosen the nose clip, take off the scarf gently and preserve it for further use
7. Keep away from fire

Principles of washing:

1. Hand wash only in warm water (up to 30 °C) using washing powder or washing soap
2. Do not use bleach, bile soap, fabric conditioner or other water softeners
3. Do not dry clean

Recommended procedure:

1. Hand wash gently. Handle the nanofiber membrane with care – just soak the filter in water and rub it in your hands
2. After washing, wring the scarf out gently by hand. Spin-drying is not recommended. Hang the scarf and let it dry freely, do not dry in a dryer
3. Do not iron or steam. Ironing or thermal straightening may cause permanent damage to the scarf or lower its efficiency